


NoviPlex Uno Card Blood Application Instructions

Visit Telimmune.com to view the full training video. For any additional questions, please request nurse or qualified personnel in the study.

DO NOT touch the sample port when depositing blood onto the card. This will contaminate your sample.




1. Open the card from the bottom. Save the sleeve to place card back in after procedure is over

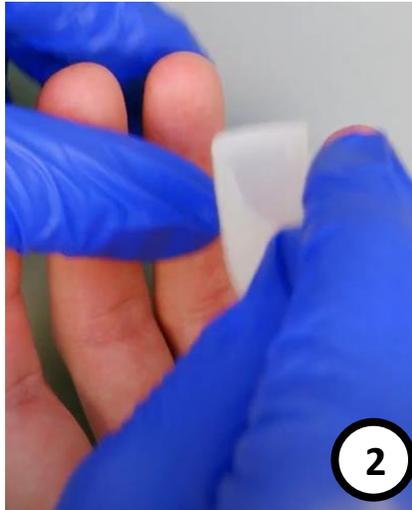


2. Thoroughly warm hands first, then clean the area to be punctured using an alcohol wipe.




3. Press lancet firmly against the skin and release lancet by pressing top button

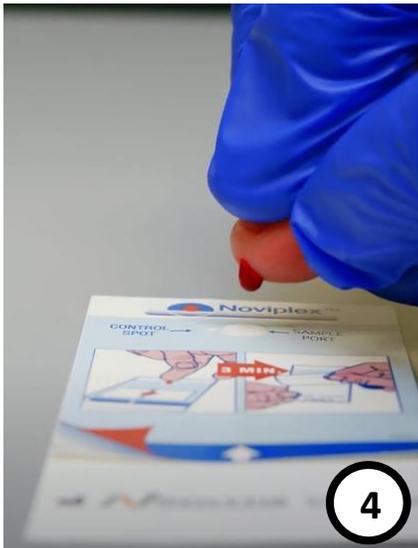

4. With palm down, apply pressure at the base of the finger towards the tip to increase size of blood drop. Keep the drop pointed towards the card (hand turned over) to keep the blood from spreading on the finger

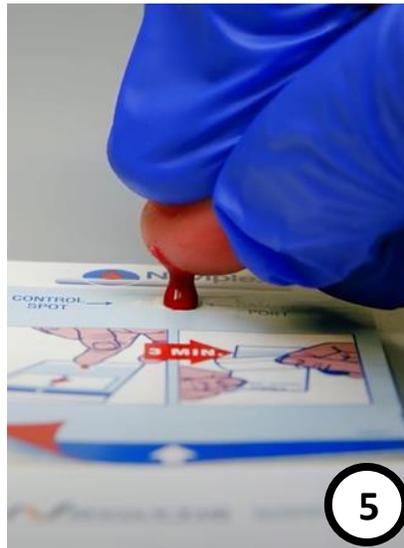

5. Gently touch blood spot to "test area" on card. Be careful to not touch the card with the finger!

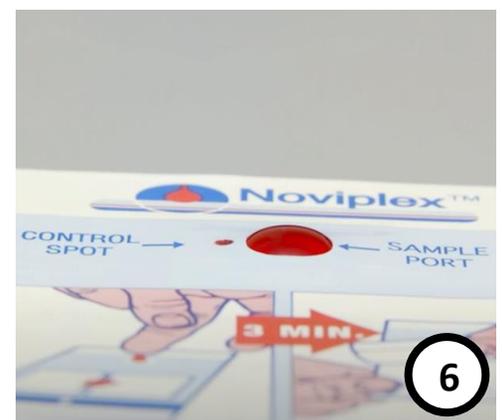

6. If the control spot does not turn red within 30 seconds, immediately add an additional drop of blood. Peel the top layer of the card at exactly three minutes following initial blood application.

Corresponding Days to Weekday

Please place the ID sticker according to the schedule below

Day 0 = Monday (Admission to clinic)

Day 1 = Tuesday

Day 2 = Wednesday

Day 3 = Thursday

Day 4 = Friday

Additional Tips for Effective Blood Collection

- Rub hands together or under warm/hot water to warm them up. This will get your blood flowing
 - o You can also move your arms around (fingers should appear pink)
- Collect blood from the dominant hand as it may be easier to get a large enough drop
- It is better to prick the sides of the finger rather than the center, this will be less painful (go for the smiley face)
- If you notice that there is enough blood to collect a second card, please do so.